



STEM CELL LABORATORY (STCL)



DOCUMENT NUMBER: STCL-EQUIP-023 FRM1

DOCUMENT TITLE:

Sysmex XN-450 QC Review Log - DAILY

DOCUMENT NOTES:

Document Information

Revision: 01

Vault: STCL-Equipment-rel

Status: Release

Document Type: STCL

Date Information

Creation Date: 19 Mar 2025

Release Date: 20 Mar 2025

Effective Date: 20 Mar 2025

Expiration Date:

Control Information

Author: WATER002

Owner: WATER002

Previous Number: None

Change Number: STCL-CCR-562

Equipment: Sysmex XN-450
SN: 13201
Location: Cell Processing

STCL-EQUIP-023 FRM1
SYSMEX XN-450 QC Review Log - DAILY

Mark Y or N by each entry and Initial QC review as performed. Month: _____ Year: _____

Record problems and corrective action on reverse side. N/A = Not Applicable NIU = Instrument Not in Use

Lot # _____ Implementation Date: _____ Lot # _____ Implementation Date: _____

Instructions for handling QC Material:

Remove Sysmex XN-L-Check QC vials from refrigerator; allow the vials to come to room temperature for ~ 15 minutes. Mix the vials by gentle end-to-end inversion until the cell button in the bottom of each vial is completely suspended.

	01	02	03	04	05	06	07	08	09	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
DAILY																															
1 st Shift																															
Level 1 QC, OK? Y or N (If N(o), complete QC trouble shooting log on reverse page with details)																															
Tech Initials																															
Level 2 QC, OK? Y or N (If N(o), complete QC trouble shooting log on reverse page with details)																															
Tech Initials																															
Level 3 QC, OK? Y or N (If N(o), complete QC trouble shooting log on reverse page with details)																															
Tech Initials																															
2 nd Shift																															
Level 1 QC, OK? Y or N (If N(o), complete QC trouble shooting log on reverse page with details)																															
Tech Initials																															
Level 2 QC, OK? Y or N (If N(o), complete QC trouble shooting log on reverse page with details)																															
Tech Initials																															
Level 3 QC, OK? Y or N (If N(o), complete QC trouble shooting log on reverse page with details)																															
Tech Initials																															

Weekly Review: (Initials/Date) Wk1 _____ Wk2 _____ Wk3 _____ Wk4 _____ Wk5 _____
Monthly Review: (Initials/Date) _____

STCL-EQUIP-023 FRM1 Sysmex XN-450 QC Review - DAILY

Date	Outlier (including the Level of QC and the Parameter)	Result obtained	Lot # / Action Taken	Tech Initials

Weekly Review: (Initials/Date) Wk1 Wk2 Wk3 Wk4 Wk5

Monthly Reviews/Date and Initial:

Instructions for STCL-EQUIP-023 FRM1 Sysmex XN-450 QC Review – DAILY

On the form, record the lot # and implementation date of that lot #; if a second lot # is implemented during a month, enter that lot # and implementation date in the second set of data fields.

It is VERY important that the QC material be handled exactly according to the instructions. This will ensure that the samples have time to equilibrate to room temperature and are mixed adequately before they are tested. Handling the QC material consistently at all times will minimize the number of variables that need to be investigated if/when parameters are out of acceptable limits.

When a QC parameter does not produce satisfactory results, corrective action /an investigation must be performed to determine whether there is an instrument malfunction, a problem with the QC material, or technical error. The investigation must be complete before reporting out any patient-related test results.

The date the problem was encountered, the outlier (including the parameter and level of QC), the result of that parameter, the Lot #, action taken, and the initials of the tech identifying the problem must be documented.

When a QC parameter does not produce satisfactory results, corrective action /an investigation must be performed to determine whether there is an instrument malfunction, a problem with the QC material, or technical error. The investigation must be complete before reporting out any patient-related test results.

QC GUIDELINES:

- IF one level of QC for any parameter(s) is (are) outside the 2SD limit, QC should be reanalyzed.
 - IF repeat QC is **within 2SD range**, patient results can be reported.
 - If the repeat QC value(s) is **outside the 2SD range**, the LJC QC data must be reviewed for all three (3) levels of QC material.
 - If there are NO previous outliers and the last five (5) or more QC values are evenly distributed around the mean, repeat QC using a fresh vial.
 - If repeat fresh QC is within 2SD, patient results can be reported.
 - If repeat fresh QC is outside 2SD, do NOT report patient results and consult senior staff for troubleshooting options.
 - If there are previous outliers OR if previous QC values are falling on one side of mean, bring this to the attention of senior staff for troubleshooting options. Do NOT report patient results until QC values fall within the 2SD range.
- IF two or more levels of QC for any parameter are outside the 2SD limit, STOP patient analysis and troubleshoot. Patient results can NOT be reported until results are within the 2SD limit.
- When the last 5 or more QC values for any parameter are found to be plotting on the same side of the mean for two or more levels of QC material, inform the senior staff. If the parameters are plotting within the 2SD limit, patient results may be reported.

Signature Manifest**Document Number:** STCL-EQUIP-023 FRM1**Revision:** 01**Title:** Sysmex XN-450 QC Review Log - DAILY**Effective Date:** 20 Mar 2025

All dates and times are in Eastern Time.

STCL-EQUIP-023 FRM1 Sysmex XN-450 QC Review Log**Author**

Name/Signature	Title	Date	Meaning/Reason
Barbara Waters-Pick (WATER002)		20 Mar 2025, 12:21:45 PM	Approved

Management

Name/Signature	Title	Date	Meaning/Reason
Barbara Waters-Pick (WATER002)		20 Mar 2025, 12:22:05 PM	Approved

Medical Director

Name/Signature	Title	Date	Meaning/Reason
Joanne Kurtzberg (KURTZ001)		20 Mar 2025, 12:25:59 PM	Approved

Quality

Name/Signature	Title	Date	Meaning/Reason
Bing Shen (BS76)	Associate Director, Quality Assurance	20 Mar 2025, 12:49:24 PM	Approved

Document Release

Name/Signature	Title	Date	Meaning/Reason
Amy McKoy (ACM93)	Document Control Specialist	20 Mar 2025, 01:31:39 PM	Approved